


Hello! Thank you for your interest in the ATRACT device! Please follow these steps to ensure optimal use of the device

- 1) Untangle the device: it should look like this right before use:

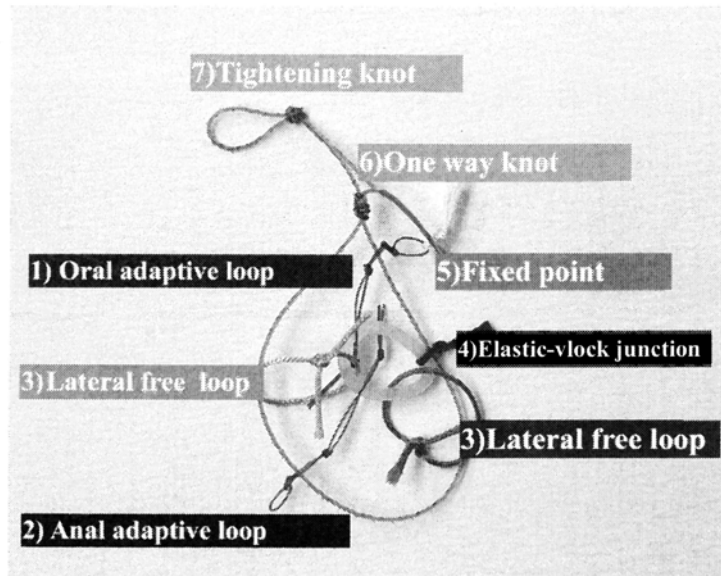

- 2) If you use a scope of diameter greater than 3.2mm, you can grab the device with a clip fixed to the oral loop and bring it to the operating site through the operating channel.
If you use a scope of 3.2mm, it might get stuck during the delivery, so it is better to first insert a clip without the device on it, and grab the device from the tip of the scope, and pull it back into the operating channel. This blocks the suction during the insertion of the scope in the patient, but it ensures the device does not get stuck.
- 3) Then follow the tutorial of use (video in the drive)
- 4) Additional tips:
 - a) Be sure to do a deep trimming before setting the device
 - b) aim for a 90° traction direction when setting the rubber band
 - c) the lateral free loops are optional, and can be set at the beginning or during the intervention when needed
 - d) when facing a large lesion, if you feel the device is very stretched, you can delay the setting of the elastic on the opposite wall of the colon and start the initial dissection phase
 - e) When tightening the device with the foreign body grasper
 - i) be sure to **place the tip of the scope as close as possible to the oral adaptive loop** as you can
 - ii) deflate the colon a bit
 - f) At the end, you can retrieve the lesion by grasping the rubber band with the foreign body grasper